

Bischoff & Bischoff

Instruction manual

Warranty section

(Please note our general business terms and conditions)

Model: B+B shower stool DH-40/L

Model serial number:*
(to be entered by dealer)

Dealer:

Date and stamp

DH-40/L



Bischoff & Bischoff GmbH


Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

www.bischoff-bischoff.com

international@bischoff-bischoff.com

Enjoy mobility.

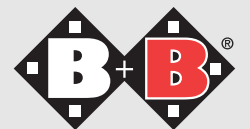

DH-40/L **(shower stool)**

Dear customer,

with the purchase of this B+B shower aid you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B+B shower aid and make your life safer.

We hope you enjoy our product.

1. Caution - safety instructions Risk analysis as per DIN EN ISO 13485

- Set all four outer feet to the same height.
- Make sure that the locking knobs are correctly seated.
- Make sure that the seat fastening screws are correctly seated.
- Make sure that the seat back is correctly seated.



- Foot section A
- Fixation screws.
- Fixation screw (longest screw)
- Fixation screws.
- Foot section B
- Locking button for height adjustment
- Screws for handles

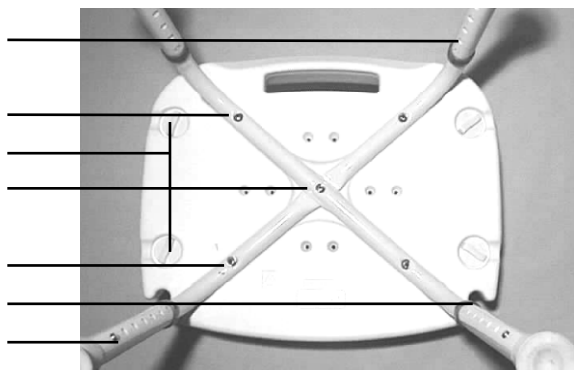


Fig. 1

1. Installing the A+B feet

Place the seat on its side with the bottom towards you (Fig.1). Take the A + B foot sections, place crosswise one on top of the other (rubber feet up, flattened sections must fit together) and fasten both foot sections with a screw (fixation screw with washer and lock washer) in the centre of the seat. Fix the centre of the shower stool with the longest screw. Now align foot sections A + B so you can fasten them with the remaining four screws (with washers).

Check that all fixation screws are tight.

2. Installing the seat back (DH-40 L only)

Press the seat back (Fig. 3) with the anatomically shaped side to the front into the holder (Fig. 2).



The bottom section of the seat back must extend approx. 5 mm from the bottom of the seat.



Fig. 2



Fig. 3

3. Adjusting the height of the seat

Both feet are height-adjustable. Press the locking knobs (Fig. 1) in and move the outer feet to the desired height.

The seat height can be adjusted to eight positions.



Make sure that the knobs are locked after adjustment.

After you have finished, slide the rubber seals over the outer feet.



All four feet must be set to the same height.

The shower stool is supplied with antislip rubber feet. Suction feet are optionally available.

4. Fastening the side handles

Insert handles into the holes from above and fix from below with the screws (Fig. 1).



Check that screws are tightly fastened

5. Fastening the shower head to the shower stool

Two shower-head holders are installed on the front of the shower stool seat to hold the shower head while showering.

6. Cleaning the shower stool

Clean the shower stool with standard mild cleaners only.

7. Technical data

Max. load capacity:	110 kg
Seat width:	50 cm
Seat height:	40 - 57 cm (78 - 95,5 cm)
Seat depth:	40 cm
Weight DH-40 (L):	4,1 kg