

Bischoff & Bischoff


Instruction manual


RL-120

RL-120 Vario



Enjoy mobility.

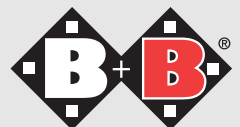

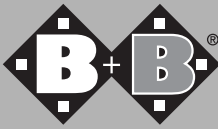

RL-120 | RL-120 Vario **(light-weight rollator)**

Dear customer,

with the purchase of this B + B walking aid you now have a top-quality German product that has given reliable service in many areas of professional use.

This instruction manual is provided to familiarise you with the use and maintenance of your B + B walking aid and to assist in making your daily life safer.

We hope you enjoy our product and the mobility that it will give you.

1. Attention - safety instructions! **(Risk analysis as per DIN EN 13485)**

- Max. load capacity 120 kg
- Do not hang bags or similar objects on the handles.
- Use your walking aid on even and horizontal surfaces only.
- Make sure that the handlebars are correctly fastened.
- Each brake works on only one wheel.
- Do not use aggressive or acidic cleaning agents.
- Have your brakes checked by the dealer every six months.

2. Indications / contraindications

The rollator is designed to assist users with difficulties in walking, e.g. for domestic use, during shopping or walking outdoors. The seat can be used for resting over long distances or pauses.

The rollator must be used on flat and solid surfaces only. The product must not be used as a ladder or to carry heavy loads.



Handle
Brake with lock
Back rest

Tray
Basket
Autolock-System
Handlebar lock

Wheel

Steering wheel

3. Unfolding

- Position the B+B rollator in front of you in the direction of travel.
- Set the parking brake (see Section 7).
- Tilt the B+B rollator towards you until the steering wheels are unloaded and press the handles down with a slight jerk.
- To lock the folding mechanism, press the seat down with the flat of your hand.


4. Folding

- Repeat the first two steps.
- Hold the rollator by the handle with one hand.
- Reach one hand through the opening on the seat and pull the locking lever of the folding lock - Autolock system - (inset above) upwards.
- Now simply pull the seat upwards.

Please make sure that parts of the body and clothing are not jammed while folding the rollator.



Fig. 1

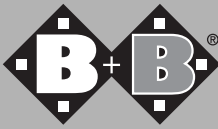

5. Attaching basket

- The B+B rollator is unfolded.
Set the parking brake (see Section 7).
- Stand in front of the B+B rollator opposite to the direction of travel.
- Hang the basket on the hooks on the front part of the frame.
- To remove the basket, simply take the handle in your hand and lift it.

6. Installing tray

- The B+B rollator is unfolded. Set the parking brake (see Section 7).
- Stand in front of the B+B rollator opposite to the direction of travel.
- Place the tray on the seat cushion from above so the four plastic feet of the tray are inserted into the holes in the seat cushion.

7. Adjusting sliding handles

- The B+B rollator is unfolded.
- Set the parking brake (see Section 7).
- Unscrew the plastic screws on the handlebars and remove the fixation screws.
- Stand behind the B+B rollator and allow your arms to hang loosely down.
- Now adjust the sliding handles so they are at the height of your wrists.
- After adjusting the handles, tighten the screws again.

The handlebars can be adjusted in one of seven positions at intervals of 2.5 cm and will not rotate. After making the adjustments, make sure that the nuts are tight.

8. Adjusting seat height (RL-120 vario only)

1. Open the locking pin, and pull it out.
2. Press the locking pins on both sides together and slide the wheel fork with the inner tube to the desired position.
3. Release the locking pins and make sure that the pin is locked in the required hole
4. Insert locking pin again and lock it.

9. Brake operation and adjustment

9.1. Service brake

The service brake is operated with the two brake levers on the handles by pulling them upwards.

9.2. Parking brake

- Press the two brake levers down to lock them.
- To release the brakes pull the two brake levers back to the centre position.

9.3. Brake adjustment


The brakes can be adjusted directly at the brake lever.

The brake is correctly adjusted when the brake lever or the cable have no play. The brake pads must not contact the wheel unless the lever is actuated.

- To adjust the brake to make it harder unscrew the knurled screw on the brake mechanism and lock it with the other nut.
- When adjusting the Bowden cable loosen the locknut (M6) first.



Fig. 2

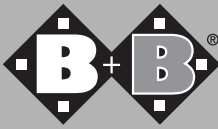

- Then the threaded sleeve is adjusted until the correct setting has been found. To prevent the setting from changing the threaded sleeve must be locked with the nut (M6).
- To adjust the brake to make it softer follow the above procedure in reverse.

10. Wheels and tyres

PU tyres are mounted on the 8" wheels.

11. Technical data

	RL-120	RL-120 Vario
Model:	light-weight rollator Aluminium	light-weight rollator Aluminium
Colour:	Silver metallic	Silver metallic
Length:	78 cm	78,5 cm
Width:	62 cm	60 cm
Seat height:	51 cm	59 - 61,5 - 64 cm
Weight:	7,4 kg (without accessories)	7,7 kg (without accessories)

Load

Basket:	3 kg	3kg
---------	------	-----

- Package: basket, tray, back rest, Instruction Manual
- Antislip handles.
- Combined service and parking brake.
- Solid aluminium construction, corrosion-resistant coating.
- Antirotation height-adjustable handles
- The rollator can be used outdoors and indoors.
- Both rollators are designed for repeated use (see B+B Service schedule for walking aids).

Warranty section

(Please note our general business terms and conditions)

Model: rollator

Serial number:*

(Please note our general business terms and conditions)

Dealer:

Date and stamp

* See the side frame for information stickers