


Bischoff & Bischoff


Instruction manual

Rollator B **Rollator B XL**



Enjoy mobility.

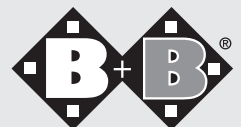

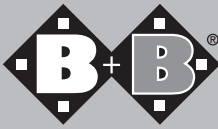

Rollator B (walking aid)

Dear customer,

with the purchase of this B+B walking aid you have acquired a high-quality German product, thousands of which are already in professional use. This instruction manual is designed to assist you with the use and maintenance of your B+B walking aid and to make your daily life safer.

We hope you enjoy our product and the mobility that it will give you.

1. Attention - safety instructions!

- Max. load capacity 120 kg, Rollator B XL 200 kg.
- Use your walking aid on even and horizontal surfaces only.
- Note minimum insertion depth of handlebars (mark).
- The push handles may only be rotated outwards by a max. of 45°. See also the safety information.
- Each brake works on only one wheel.
- Do not use aggressive or acidic cleaning agents.
- Have your brakes checked by the dealer every six months.






- Handle
- Parking brake
- Handle bar
- Tray
- Screw of the handle bar
- Basket
- Locking lever
- Walking stick holder
- Steering wheel
- Drive wheel

2. Unfolding

Position the B+B rollator in front of you in the direction of travel.

- Set the parking brake (see Section 7).
- Tilt the B+B rollator towards you until the steering wheels are unloaded and press the handles down with a slight jerk.
- To lock the folding mechanism, press the cross bar down with the flat of your hand and lock the locking hook.

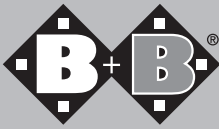

3. Folding

- Repeat the first two steps.
- Release the locking lever in the middle of the rollator
- Hold the rollator by the handle with one hand.
- Hold the cross bar and pull it straight upwards.
- Please make sure that parts of the body and clothing are not jammed while folding the rollator.

4. Attaching basket

- The B+B rollator is unfolded. Set the parking brake (see Section 7).
- Stand in front of the B+B rollator opposite to the direction of travel.
- Hang the basket on the hooks on the front part of the frame.
- To remove the basket, simply take the handle in your hand and lift it.

5. Installing tray

- The B+B rollator is unfolded. Set the parking brake (see Section 7).
- Stand in front of the B+B rollator opposite to the direction of travel.
- Place the tray on the seat cushion from above so the four plastic feet of the tray are inserted into the holes in the seat cushion.
- The push handles may only be rotated outwards by a max. of 45°. See also the safety information.

6. Adjusting sliding handles

- The B+B rollator is unfolded.
- Set the parking brake (see Section 7).
- Unscrew the plastic screws on the handle bars until the handles move freely.
- Stand behind the B+B rollator and allow your arms to hang loosely down.
- Now adjust the sliding handles so they are at the height of your wrists.
- After adjusting the handles, tighten the screws again. Note the minimum insertion depth of the handlebars. The mark on the handlebar (minimum) must not be visible (below the expansion slots in the frame tube).


7. Brake operation and adjustment

7.1 Service brake

The service brake is operated with the two brake levers on the handles by pulling them upwards.

7.2 Parking brake

- Press the two brake levers down to lock them.
- To release the brakes pull the two brake levers back to the centre position.

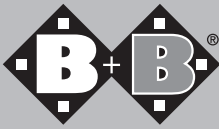

7.3. Brake adjustment

The brakes can be adjusted directly at the brake lever. The brake is correctly adjusted when the brake lever or the cables have no play. The brake pads must not contact the wheel unless the brake lever is actuated.

- To adjust the brake to make it harder unscrew the knurled screw on the brake mechanism and lock it with the other nut.
- When adjusting the Bowden cable loosen the locknut (M6) first.
- Then turn the threaded sleeve until the adjustment is correct. To prevent the setting from changing the threaded sleeve must be locked with the nut (M6).
- To adjust the brake to make it softer, carry out the above procedure in reverse.

8. Wheels and tyres

Your rollator is fitted with PU wheels size 200 x 50 mm.

9. Technical data

Model:	Rollator B	Rollator B XL
Colour:	Silver metallic	Silver metallic
Length:	68 cm	64 cm
Width:	61 cm	71 cm
Seat height:	61 cm	63 cm
Folded:	24 cm	24 cm
Weight:	10.1 kg	12.9 kg

- With walking stick holder, basket and tray.
- Anatomically shaped handles or rubber handles.
- Combined service and parking brake.
- Solid steel construction, corrosion-resistant coating.
- Non-rotating handle bars, infinitely adjustable from 78 to 94 cm.
- PU tyres 200 x 50 mm.

Warranty certificate

(Please note our general business terms and conditions)

Model: B+B Rollator

Serial number:*

(Please note our general business terms and conditions)

Dealer:

Date and stamp

Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

www.bischoff-bischoff.com

international@bischoff-bischoff.com