

Bischoff & Bischoff

Instruction manual

Warranty certificate

(Please note our general terms and conditions)

Model: B+B TS-1 commode wheelchair

Serial number: *
(to be entered by dealer)

Dealer:

Date and stamp

TS-1



Bischoff & Bischoff GmbH


Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

www.bischoff-bischoff.com

international@bischoff-bischoff.com

Enjoy mobility.

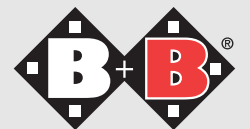

TS-1 (commode wheelchair)

Dear customer,

with the purchase of this B+B commode wheelchair you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B + B commode wheelchair and make your life safer.

We hope you enjoy our product and are more mobile with it.

1. Attention - safety instructions! (Risk analysis as per DIN EN 13485)

- Max. load capacity 120 kg
- Never use the commode wheelchair without an assistant.
- Use your walking aid on even and horizontal surfaces.
- Always lock both brakes when using your wheelchair.
- Do not lean out to the side. Danger of tipping!
- Make sure that the seat-back screws are tight.
- The commode wheelchair must not be used by unauthorised persons (particularly children).

Commode wheelchairs are recommended if the patient cannot use or access toilette because of the disability.



- Back cover
- Locking pin for Arm rest
- Locking lever for leg supports
- Leg supports
- Front steering wheel, unbraked
- Removable seat back
- Adjustable arm rest with padding
- Seat cushion
- Fixation screw for back rest
- Commode with cover
- Back steering wheel, braked



Fig. 1

2. Parking brake

The two rear steering wheels with brakes prevent the wheelchair from moving accidentally.

- Setting brakes: press brake lever down to the limit stop with your foot (Fig. 2)
- Releasing brakes: pull the seat upwards with your toe (Fig. 2).

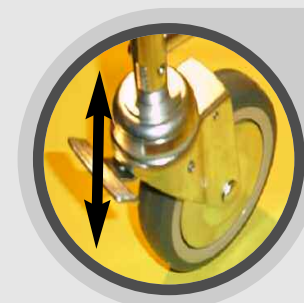


Fig. 2

3. Leg supports

To make it as easy as possible to enter and leave your B + B commode wheelchair, the leg supports can simply be swung to the side out of the way. It is not necessary to remove them.

- Press the locking lever slightly inwards (Fig. 3) to release the lock.
- Swing the leg supports outwards.
- If necessary, fold the foot plates up first.
- To restore the original status, just swing the leg supports forward and lock them in place.
- To remove the leg supports proceed as follows:
 - Press the locking lever slightly inwards (Fig. 3) to release the lock.
 - Swing the leg support outwards.
 - Pull the leg support out upwards.

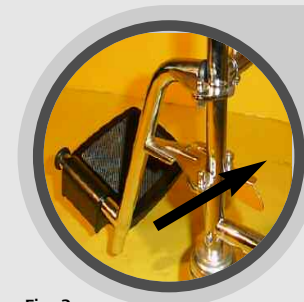


Fig. 3



Danger of accident!

Be careful not to stand on the foot plates and/or the foot supports when standing.



Fig. 4

4. Commode seat

Removing commode seat:

- Pull front edge of seat up from the base.
- Lift commode seat upwards.

Positioning commode seat:

- Slide back edge of seat with the open corners to the back.
- Press front rounded edge of seat down to the stop. Seat must lock into position.
- Make sure that it is locked.

5. Arm rests

The arm rests can be folded down.

- To fold them down, pull out the locking button and fold the arm rest down (Fig. 4).
- To return it to its original position fold the arm rest up to its original position until you hear the locking pins click into place.



Fig. 5



Attention!

Make sure that there are no persons behind your commode wheelchair when adjusting the arm rests.

8. Technical data

Seat width:	45 cm
Seat depth:	44 cm
Length:	96 cm
Height:	97 cm
Width:	56 cm
Seat to arm rests:	22 cm
Seat back rest:	34 cm
Max. user weight:	120 kg
Weight:	15,3 kg