

Bischoff & Bischoff

Instruction manual

Warranty section

(Please note our general terms and conditions)

Model: B+B commode seat TS-200

Model serial number:*
(to be entered by dealer)

Dealer:

Date and stamp

TS-200



Bischoff & Bischoff GmbH


Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

www.bischoff-bischoff.com

international@bischoff-bischoff.com

Enjoy mobility.

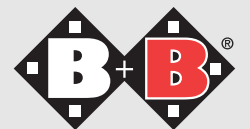

TS-200 (commode seat)

Dear customer,

with the purchase of this B+B commode seat you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B+B commode seat and make your life safer.

We hope you enjoy our product and are more mobile with it.

1. Attention - safety instructions! (Risk analysis as per DIN EN 13485)

- Max. user weight 120 kg!
- The commode seat should only be used indoors and with the aid of an assistant.
- Push the seat on even and horizontal surfaces only.
- Always lock both brakes when at a standstill.
- Do not lean out to the side. Danger of tipping!
- Make sure that the seat-back screws are tight.
- The commode seat must not be used by unauthorised persons (particularly children).

Commode wheelchairs are recommended if the patient cannot use or access toilette because of the disability.



- Back cover
- Adjustable arm rest with padding
- Locking pin for arm rest
- Seat cushion
- Fixation screw for back rest
- Locking pin for leg support
- Commode with cover
- Leg supports
- Back steering wheel, braked
- Front steering wheel, unbraked



Fig. 1

2. Parking brake

The two rear steering wheels with brakes prevent the commode seat from moving accidentally.

- Locking: press brake lever 1 down to the limit stop with the foot (Fig. 2)
- Releasing: press lever 2 to release the brake (Fig. 2).

3. Leg supports

To make it as easy as possible to enter and leave your B+B commode seat, the leg supports can simply be swung to the side out of the way. It is not necessary to remove them.

- Pull the locking pin slightly out (Fig. 3) to open the lock.
- Swing the leg supports outwards.
- If necessary, fold the foot plates up first.

To restore the original status, just swing the leg supports forward and lock them in place. To remove the leg supports proceed as follows:

- Pull the locking pin out (Fig. 3) to open the lock.
- Swing the leg support outwards.
- Pull the leg support out upwards.



Danger of accident

Be careful not to stand on the foot plates and/or the foot supports when getting out of the commode seat. Danger of accident!



Fig. 2



Fig. 3



Fig. 4

4. Commode seat

Removing commode seat:

- Pull front edge of seat up from the base.
- Lift commode seat upwards.

Positioning commode seat:

- Slide back edge of seat with the open corners to the back.
- Press front rounded edge of seat down to the stop. Seat must lock into position.
- Make sure that it is locked.

5. Arm rests

The arm rests can be folded back.

- To fold them back pull out the locking button and fold the arm rest back (Fig. 4).
- To return it to its original position fold the arm rest forward to its original position until you hear the locking pins click into place.



Make sure that the arm rests are locked in position.

8. Technical data

Seat width:	45 cm
Seat depth:	44 cm
Seat to arm rests:	24,5 cm
Seat back rest:	33 cm
Depth:	94,5 cm
Height:	97,5 cm
Width:	53,5 cm
Max. user weight:	120 kg
Weight:	15,4 kg