

# Bischoff & Bischoff

Instruction manual

## Warranty certificate

(Please note our general terms and conditions)

**Model:** TS-Care commode seat

**Serial number:\***  
(to be entered by dealer)

**Dealer:**

Date and stamp

## TS-Care



### Bischoff & Bischoff GmbH


Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 72 48/92 09-2 • Fax: +49 72 48/92 09-3

[www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

[international@bischoff-bischoff.com](mailto:international@bischoff-bischoff.com)

*Enjoy mobility.*

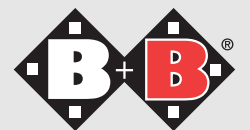

## TS-Care (Commode seat without wheels)

### Dear customer,

with the purchase of this B+B commode seat you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B+B commode wheelchair and make your life safer.

We hope you enjoy our product.

### 1. Attention - safety instructions (Risk analysis as per DIN EN ISO 13485)

- Max. user weight 120 kg
- Never use the commode chair without an assistant.

### 2. Indication ranges

Poor body control, vertigo, functional failures of the lower extremities, stiff hip and spine, people without hands, quadriplegics, arm injuries with significant disorders of gripping functions.

### 3. Delivery

Your TS-Care is delivered completely assembled.



#### 4. General overview

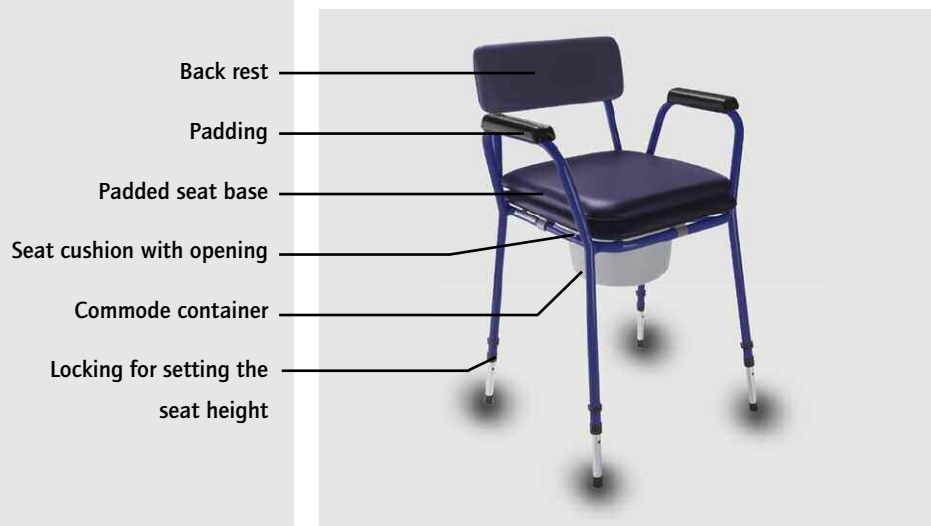


Fig. 1

#### 5. Seat

The seat cushion can be removed for easy cleaning of the chair. Before every use make sure that the seat is fixed to the frame (when the three locking clamps are obviously locked).



**Caution!**  
Danger of becoming jammed.

#### 6. Commode container

The commode container is easy to replace or empty by removing from the back of the commode seat from its holder.

With the cover closed the commode container is odour-free.

#### 7. Seat height adjustment

To adjust the seat height of the commode seat, release the locking pins on all four feet and adjust them to the desired position. The seat height can be adjusted to five positions.



**Caution!**  
Set all four feet to the same height.



Fig. 2

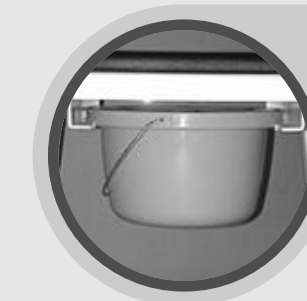


Fig. 3

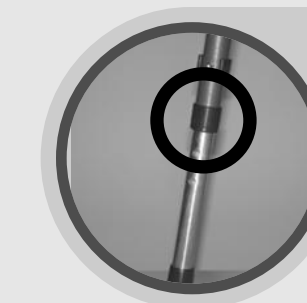


Fig. 4

## 8. Care

Your B+B commode seat will retain its attractive appearance for years with proper care. Use a standard cleaning agent or a disinfectant for cleaning stubborn dirt.

Do not use aggressive or acidic cleaning agents for cleaning.

## 9. Technical data

- Seat width: 40 cm
- Seat depth: 42 cm
- Seat height: 50-60 cm (5x adjustment)
- Arm rest height: 19,5 cm (16 cm with seat base)
- Seat back height: 32 cm (29 cm with seat base)
- Overall dimensions (L/W/H): 54-55 cm/ 56.5 cm/81.5-91.5 cm
- Overall weight: 6 kg (7,7 kg with seat base)
- Max. user weight: 120 kg

## 10. Warranty

A warranty claim can only be accepted if the product is used under the specified conditions and for the specified purposes. The product must be returned in the original packaging.