


Bischoff & Bischoff


Instruction manual

TSE-Easy 10



Enjoy mobility.

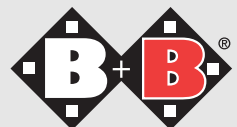

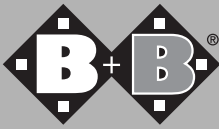

TSE-Easy 10 (Toiler seat raiser)

Dear customer,


with the purchase of this B+B toilet aid you are now the owner of a high-quality German product, thousands of which are already in professional use.

This instruction manual will familiarise you with the use and maintenance of your B + B toilet aid and make your life safer.

We hope you enjoy our product.

1. Attention - safety instructions!

- Max. load capacity 200 kg
- The load capacity is not based on the cover of the commode seat. The cover is not suitable for use as a seat.
- The toilet seat raiser is not suitable for continuous use as a seat.
- Make sure that the fixation screws are tight.

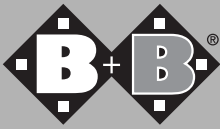

Commode cover

Fixing clamp

Toilet seat



TSE-Easy 10

2. Indication range

The use of the toilet seat raiser is indicated for patients with poor body control, vertigo, significant functional disorders of the lower extremities, stiffness in hips and spine and for people with no hands, quadriplegics or people with arm disorders with significant disorders of the gripping functions.

3. Assembly

- Insert the blue plastic screw into the fixing clamp. Make sure that the groove in the screw is fixed in the bulge in the fixing clamp.- Insert the assembled fixing elements into the groove of the seat base (Fig. 2)
- Screw the plastic screw lightly down into the thread in the seat base.

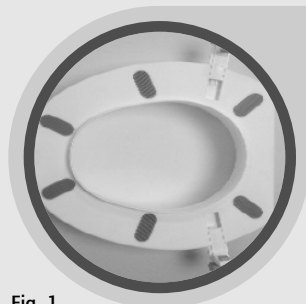



Fig. 1



Fig. 2



Fig. 3

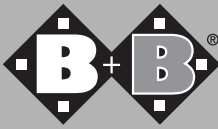

4. Installation

- Place the TSE-Easy 10 on your toilet.
- Fixed the TSE-Easy 10 to the toilet bowl with the fixing clamps.

Make sure that the screws are tight to hold the TSE-Easy 10 firmly on the toilet.

5. Cleaning and disinfection

Instructions for reuse: clean and disinfect the TSE-Easy 10 with standard cleaners and disinfectants only. Do not use aggressive cleaning agents.

6. Technical data

Seat width:	36 cm
Seat depth:	39 cm
Toilet seat raiser:	10 cm
Overall height:	16 cm
Overall width:	40 - 48 cm
Overall depth:	40 cm
Weight:	1,5 kg
User weight:	200 kg
Accessories:	2 clamps

Warranty certificate

(Please note our general business terms and conditions)

Model: TSE-Easy 10

Serial number:*

(to be entered by dealer)

Dealer:

Date and stamp

* See the side frame for information stickers

Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 (0)700/6000 7070 • Fax: +49 (0)700/6000 8080

www.bischoff-bischoff.com

info@bischoff-bischoff.de

